HOSPITAL STUHL - HOSPITAL ARMCHAIR

System "long life" - "Long life" system

STEPHEN HEASBY

STEPHEN H EASY

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Oneness and previewed use

The **EASY** chair is used for incoming emergency admissions, for short hospitalization and patient transportation.

The structure has three sections, two of them electrically powered (with accessory AC12). The chair has independent movement of the backrest and leg rest via a mains rechargeable battery.

The „long-life“ system provides energy for a normal days use.

The movement of the sections are made via low voltage motors controlled by an handset easy accessible to the patients.

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Frame

The anti-tilting frame is made out of mild steel with a high quality paint finish, or stainless steel. The chair complies with EN 60601-1/IEC 601-1. All critical parts, under wear and mechanical stress are made of stainless steel AISI 304 to satisfy the safety limits of EN60601-1/ IEC 601-1. Access to all parts of the chair make cleaning and disinfection simple. The base has four 150 mm wheels. The rear wheels are provided with a central brake, and the front wheels are directional wheels to facilitate the maneuverability of the chair. The backrest is provided with 2 handles.

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Antibacterial upholstery

*The upholstery is completed with cleanable synthetic leather (flame retardant fabric Class 1IM) which is certified SANITIZED® with antibacterial and antimycotic properties, and **lack of phthalates complying to the Directive 2007/47/CE**. There are three independent cushions composed of expanded polymeric foam made out of polyether (class 1IM), non-deformable and with both high density (30/40Kg/m³) and lift (5,2Kpa). These specifications allow good posture, proper support and safety for the patient. All materials have an alveolar cell structure to allow air circulation. The mattress incorporates a permanent bacteriostatic compound.*

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20600

00020

Fabric is available in the following colours

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H: Full Hygienic

All the surfaces are easy to inspect. The cushions, the frame and the HPL support are easy to clean effectively with commonly used disinfectants.

The model made out of steel, the plant used is enriched with silver ion that has antibacterial properties. The model made out of stainless steel, the frame is sandblasted with ceramic. Thanks to this treatment, the surface are cleaned of all mechanical working residual, maintaining the chemical-physical features.

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Functions

The leg rest is composed of two hinged joints. The lower part can be tilted at 90° degree and becomes a comfortable support for the feet. The opening and the closing of this part are made through slight foot pressure.

The mechanism of the **EASY** chair reduces the risks of back injury if compared with other chairs on the market (like wheel chairs), where

the footrest, sited near the floor, is operated through the hand. This mechanism is easy to use and strong.

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Armrests

The chair is equipped with 2 armrests with adjustable height. The lowest height is parallel to the seat, enabling side access to the chair and seat wider when needs. The armrests are rotating, adjustable in height, and completely removable. The armrests are designed to withstand to dynamic stress.

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